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Simple Locking IntraMedullary (SLIM) System

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
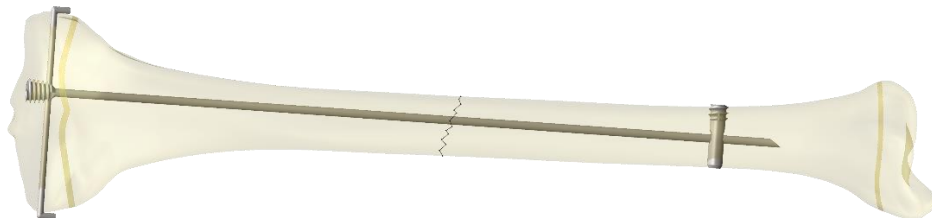
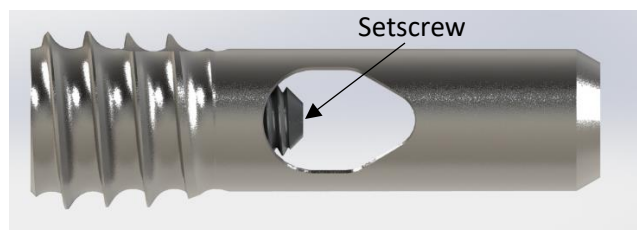
The Bullet Screws have been designed as an optional uni-cortical screw for distal locking. The SLIM rod is passed through the centered cavity of the Bullet Screw, distal fixation is achieved by applying compression onto the shaft with a set screw.

The Bullet screw is available in length ranging from 16 to 60mm in 4mm increments. It is cannulated for guidance. The Bullet screw is provided in two sizes Ø4.8mm and Ø6.0mm which corresponds to the diameter of the shaft. The size of the shafts has been minimized, to obtain the smallest incision, while maintaining a mechanical strength.

2 – ADDITION OF UNI-CORTICAL PEGS FOR PROXIMAL AND DISTAL FIXATION

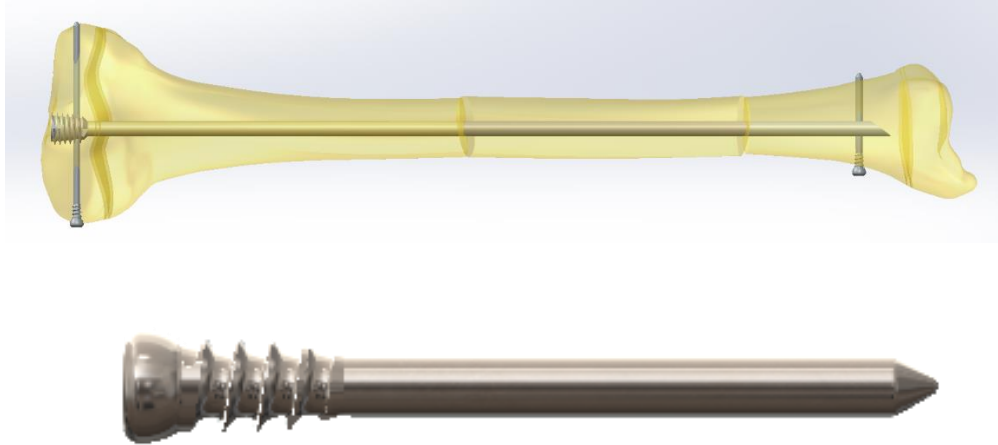The Locking Peg System has been added to allow proximal and distal fixation through this interlocking pins as shown below. The Locking Pegs are compatible with Proximal and distal holes of the SLIM Nails.

The Locking Pegs are available in three diameters: 2.0mm, 2.4mm and 2.8mm. It is a solid shaft implant available in the same lengths from 16 to 60mm in 4mm increments. The chamfered tip allows guided insertion through the prepared hole into the far cortex. The cortical thread are design to allow proper fixation into the lateral cortex. The torx T6 or T8 drive feature ensures proper torque application onto the implant.